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Normative requirements

Second category personal equipment that conforms to the Regulation UE 2016/425 requirements for the design and manufacture of the PPE;

UNI EN 166:2004 - Personal eye protection

Specifications, CE 765/2008 - Graphic signs - Colors and safety signs
Registered safety signs.

PPE manufactured according to the laws:
UNI EN 166:2004 UNI EN 167:2001 UNI EN 168:2001

Instructions for Use

NOTIFIED BODY

The protective visors and protective glasses, are category II PPE for this reason they need the issued UE that conforms with the Regulation (UE) 2016/425 from a notified body.

Larident SRL collaborated with the notified body MTIC INTERCERT
Via G. Leopardi, 14 - 20123 Milan (MI) - Italy

WARNING: wear PPE before exposing yourself to risk

Marking:

Shield marking: L 1 S CE

L (Larident identification) - 1 (optical class) - S (increased robustness) - CE (marking)

Frame marking: L EN 166 3 S CE

L (Larident identification) - EN 166 (number of standard applied)
3 (protection against liquid sprays) - S (increased robustness) - CE (marking)

Symbol read the instructions
carefully

CE conformity marking

The foreseen duration of the
product is 5 years, while
un-opened and packaging
intact, if kept away from
direct heat.

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)
16030 Tribogna (Genova)
Tel. +39 0185 939481 - Fax. +39 0185 938595
info@larident.it - www.larident.it

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)
16030 Tribogna (Genova)
Tel. +39 0185 939481 - Fax. +39 0185 938595
info@larident.it - www.larident.it

EN

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The protective visor is classified as PPE class II, it protects the face from drops/sprays of water and saliva and reduces risks of infection.
It's a cheaper visor with its sponge and elastic in PET, with double protective film.
The elastic is very comfortable and adjustable for every face shape.

R4 VISOR

DESCRIPTION

METHOD OF USE

- Use as a face protection from drops of saliva or water.
- Use by single person.
- Can be worn in combination with other non-connected PPE, such as FFP masks or eyeglasses.
- The product is supplied already assembled, wear it so that the sponge and the elastic are comfortable for the face.

HOW TO WEAR: INSTRUCTIONS FOR USE

Wear the PPE as if they were eyeglasses, checking that the screen covers the entire face of the user and that the upper part of the visor is adherent to the forehead.

For a correct positioning of the visor, check that the nose pad is snug, adjusting the lace according to the size of the user's head.

Once you have verified that the field of view is good and does not blur or impair your vision, the PPE will be worn correctly.

**ATTENTION: IF THE VISOR IS SCRATCHED OR FOGGED,
IT MUST BE REPLACED WITH A NEW ONE.**

R4 VISOR

DESCRIPTION

Warnings

The materials used to compose our visors can cause allergies to very sensitive skin. Be careful and handle with care.

Maintenance

Do not use alcohol solutions, they can cause scratches or fogging.

CLEANING

For the screen use non-alcoholic disinfectant lotions based on soaps or suitable detergents, the same for the frame. After washing, remove all remnants of the detergent under running water.

RECOMMENDATIONS

- Carefully inspect the PPE before each use, and do not use it if any part shows signs of evident wear or malfunction.
- Do not tamper with the PPE.
- Follow the assembly instructions indicated in the previous paragraph.
- If the PPE has suffered damage or scratches to the screen, it must always be replaced.

Normative requirements

Second category personal equipment that conforms to the Regulation UE 2016/425 requirements for the design and manufacture of the PPE;

UNI EN 166:2004 - Personal eye protection

Specifications, CE 765/2008 - Graphic signs - Colors and safety signs
Registered safety signs.

PPE manufactured according to the laws:
UNI EN 166:2004 UNI EN 167:2001 UNI EN 168:2001

Instructions for Use

NOTIFIED BODY

The protective visors and protective glasses, are category II PPE for this reason they need the issued UE that conforms with the Regulation (UE) 2016/425 from a notified body.

Larident SRL collaborated with the notified body MTIC INTERCERT
Via G. Leopardi, 14 - 20123 Milan (MI) - Italy

WARNING: wear PPE before exposing yourself to risk

Marking:

Shield marking: L 1 S CE

L (Larident identification) - 1 (optical class) - S (increased robustness) - CE (marking)

Frame marking: L EN 166 3 S CE

L (Larident identification) - EN 166 (number of standard applied)
3 (protection against liquid sprays) - S (increased robustness) - CE (marking)

Symbol read the instructions
carefully

CE conformity marking

The foreseen duration of the
product is 5 years, while
un-opened and packaging
intact, if kept away from
direct heat.

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