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Tel +39 0185 939481 - Fax +39 0185 938595

info@larident.it - www.larident.it

# MANUFACTURER INFORMATION ON REUSING OF PPE ACCORDING TO UNI EN 166:2004

**MANUFACTURER:** Larident Srl

**REF:** R4

---

**WARNING NOTICE:** PROTECTIVE VISOR WITH SPONGE PACKAGE,  
ONLY AND EXCLUSIVELY PROTECTS US FROM DROPS/SPRAYS OF WATER AND SALIVA

---

**Assembling instructions:** the product is supplied already assembled

**Storage and transport:** no special requirement

**Cleaning:** clean the shield with neutral soap so it does not get scratched

**Disinfection:** use suitable disinfectant solutions

**Maintenance:** no special requirement

**Sterilization:** the product is not autoclavable

**Control/functional check:** check the possible damage and possible distortion

**Storage:** no special requirement

The above-mentioned instructions have been validated as SUITABLE for preparation of personal protection equipment (PPE) and its re-use by the medical device manufacturer.

It is the reprocessor's responsibility, and personnel involved, that the actual reprocessing achieves the required results with the equipment and materials applied in the reprocessing.

Normally validation and routine checks are necessary.

Furthermore, the reprocessor should evaluate any deviation from the provided instructions regarding efficiency and possible adverse consequences.

**LARIDENT S.r.l.**

Via Lamaneigra 12R (Ponte Rimassa)

16030 Tribogna (Genova) - ITALY

Tel +39 0185 939481 - Fax +39 0185 938595

info@larident.it - www.larident.it