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Normative requirements

Second category personal equipment that conforms to the Regulation UE 2016/425 requirements for the design and manufacture of the PPE;

UNI EN 166:2004 - Personal eye protection

Specifications, CE 765/2008 - Graphic signs - Colors and safety signs
Registered safety signs.

PPE manufactured according to the laws:
UNI EN 166:2004 UNI EN 167:2001 UNI EN 168:2001

Instructions for Use

NOTIFIED BODY

The protective visors and protective glasses, are category II PPE for this reason they need the issued UE that conforms with the Regulation (UE) 2016/425 from a notified body.

Larident SRL collaborated with the notified body MTIC INTERCERT
Via G. Leopardi, 14 - 20123 Milan (MI) - Italy

WARNING: wear PPE before exposing yourself to risk

Marking:

Shield marking: L 1 S CE

L (Larident identification) - 1 (optical class) - S (increased robustness) - CE (marking)

Frame marking: L EN 166 3 S CE

L (Larident identification) - EN 166 (number of standard applied)
3 (protection against liquid sprays) - S (increased robustness) - CE (marking)

Symbol read the instructions
carefully

CE conformity marking

The foreseen duration of the
product is 5 years, while
un-opened and packaging
intact, if kept away from
direct heat.

OTHER
LANGUAGES

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)
16030 Tribogna (Genova)
Tel. +39 0185 939481 - Fax. +39 0185 938595
info@larident.it - www.larident.it

EN

R5 VISOR INSTRUCTIONS

The face shield is classified as category II PPE and allows you to protect the face from splashes, water droplets and saliva and to reduce the risk of contagion.

Very comfortable thanks to the front foam rubber piece and the weight of only 40 g, features that make it wearable for hours safely and without hassle.

It can be used in combination with prescription glasses.

Polycarbonate protective screen with absolute transparency, replaceable quickly and easily.




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




Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)
16030 Tribogna (Genova)
Tel. +39 0185 939481 - Fax. +39 0185 938595
info@larident.it - www.larident.it

R5 VISOR

ASSEMBLY INSTRUCTIONS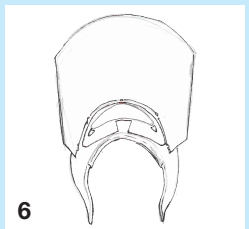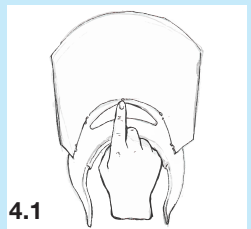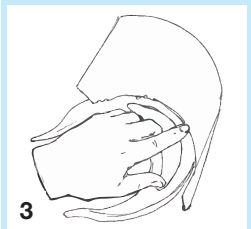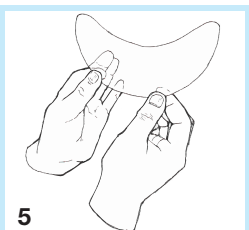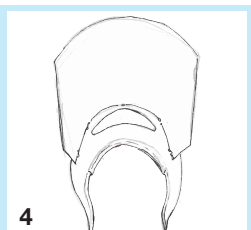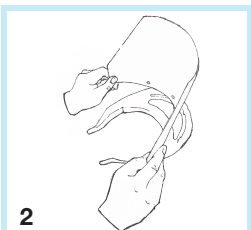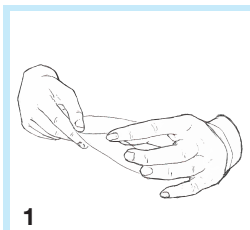

1
Turn the frame so that the pins and guides are facing the user's face

2
Take the visor by flexing it slightly. Starting from the center, first insert the screen into the highest guide, and then slide it into the lowest one; after this operation push the central hole in the central pin (horizontal) so that the PPE begins to assume a circular shape

3
At this point, move the visor to slide in the outermost guides and then push the hole into the pin (horizontal) with your fingers

4
Insert the visor in the last two guides and then push the hole into the pin (horizontal)

5
If necessary use film to cover the ventilation hole for added protection

6
For greater safety, mount the slotted elastic inserting it into the most comfortable hole

7
The visor is ready

ATTENTION: PROTECTS FROM WATER SPRAYS ONLY

The film and visor can be cleaned with a non-alcoholic disinfectant cloth

R5 VISOR

DESCRIPTION

Warnings

the materials used to compose our visors can cause allergies to very sensitive skin, be careful and handle with care.

Maintenance

do not use alcohol solutions, they can cause scratches or fogging of visors and goggles. If the visor is considered scratched or fogged by the user, it must be replaced with a new one.

CLEANING

For the screen use non-alcoholic disinfectant lotions based on soaps or suitable detergents, the same for the frame. After washing, remove all remnants of the detergent under running water.

METHOD OF USE

- Use as a face protection from saliva droplets and water droplets.
- Use indoors and outdoors with max ambient temperatures below 50 C ° and min above -10 C °.
- Use by one single person.
- Can be worn in combination with other non-connected PPE, such as FFP masks or eyeglasses.

HOW TO WEAR: INSTRUCTIONS FOR USE

Wear the PPE as if it were a helmet by tightening the elastic according to the size of your head. Once worn, check that the screen broadly covers the right and left sides of the face, the forehead and the part up to the chin. Only after checking these steps can you be sure you have worn the R5 visor correctly.

RECOMMENDATIONS

- Carefully inspect the PPE before each use, and do not use it if any part shows signs of evident wear or malfunction.
- Do not tamper with the PPE.
- Follow the assembly instructions indicated in the previous paragraph.
- If the PPE has suffered damage or scratches to the screen, it must always be taken out of use and replaced.

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