MINISABBIATRICE W1

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Tel +49 (203) 99 269-0 - Fax +49 (203) 29 92 83

www.hagerwerken.de

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WARNING: EXCLUSIVELY FOR EXTRA ORAL USE

Sandblasters significantly increase surface adhesion through sandblasting.

- it is ideal in helping with adhesion techniques
- it significantly reduces non-adhesion risks
- it is very easy to use

The system has a special valve that creates a continuous, constant and adjustable flow of abrasive material without interruptions and pulsations.

It optimizes the retention and adhesion of various dental materials such as metals, noble metals, porcelain, composite and resin.

HOW TO INSTALL THE SANDBLASTER

A compressor of 4/8 atmospheres is necessary. Dehydrated air is not necessary even if advised because wet oxide particles could join together and create small problems.

Complete with special connection for dental unit KAVO, BIEN AIR, W&H, SIRONA OR NSK.

MINI SANDBLASTER - instructions for use -

- the sandblaster's container must be loaded up to $\frac{3}{4}$ with "aluminum oxide"
- the aluminum oxide must be dry and clean
- the sandblaster is usually held like a pen using the thumb to push the button that controls the dust
- the pointed end must be positioned 2-10mm from the surface to be treated
- to get the retention desired you can pass over the surface to be treated repeatedly, more than once
- the standard pointed end is curved at 60° and included in all the packages (cod. H401-086)
- if the sandblaster is bunged up with dirty, humid dust, you need to cover the pointed end with a finger and push the button quickly a few times.

SPARE PARTS:

- W21 CONNECTION FOR DENTAL UNIT KAVO
W22 CONNECTION FOR DENTAL UNIT BIEN AIR UNIFIX
W23 CONNECTION FOR DENTAL UNIT NSK
W24 CONNECTION FOR DENTAL UNIT SIRONA
W25 CONNECTION FOR DENTAL UNIT W&H
- W41 ALLUMINUM OXIDE 50 MICRON GR.500
W44 ALLUMINUM OXIDE 90 MICRON GR.500

DISTRIBUTORE:

Larident S.r.l.
Via Lamaineigra 12R (Ponte Rimassa)
16030 Tribogna (Genova) - ITALY
Tel +39 0185 939481 - Fax +39 0185 938595
info@larident.it - www.larident.it

PRODUTTORE:

 **Hager & Werken GmbH & Co. KG**
Ackerstr. 1,
47269 Duisburg, Germany
Tel +49 (203) 99 269-0 - Fax +49 (203) 29 92 83
www.hagerwerken.de

IT

Guarantee Period: the manufacturer guarantees that the product has all the guaranteed characteristics. The guarantee period starts on the delivery date of the product for twelve months. These defects and damage are excluded from the guarantee: wear and tear, normal use, incorrect use, operating errors, customer not paying attention while using the product, connected to inadequate power sources, things like fire, lightning, damage from humidity etc.,. In the case of justified claims for damage the manufacturer has the right to repair, replace or reduce the selling price of the product.

MANUAL OF INSTRUCTIONS

Limitation of use: Frequent use only has a small impact on this instrument although its duration mainly depends on its usage and damage caused through usage.

Internal Cleaning: Unscrew the dust container. Keep the nozzle in front closed and set off the release knob so the residue dust is expelled from the suction pipe. Remove the connected tube.

External Cleaning: not suitable for mechanical cleaning. DO NOT immerse in water. Clean the outside with a damp cloth and an appropriate detergent.

Storing and transport: no special requirement.

Disinfecting: not suitable for mechanical disinfecting. DO NOT immerse in a tray with disinfectant. Clean the outside with a disinfectant wipe.

Maintenance: no special requirement.

Packaging: case with complete sandblaster plus sand useful for the first use.

Sterilization: possible steam sterilization at 121°C

Dust Container: for 15 minutes

Checking/checking it works: visual checking for damage, wear and tear and deformation.

The instructions mentioned above must be validated as SUITABLE by the manufacturer. It's the user's responsibility that the product achieves the required results. Normally routine checks are sufficient. The user's task to assess any deviation from the instructions given regarding efficiency and possible adverse consequences.

WARNING: The product must be clean, disinfected and sterilized before its first use and after every use that follows, according to the attached instructions.

DISTRIBUTORE:

Larident S.r.l.

Via Lamaineigra 12R (Ponte Rimassa)
16030 Tribogna (Genova) - ITALY
Tel +39 0185 939481 - Fax +39 0185 938595
info@larident.it - www.larident.it

PRODUTTORE:

Hager & Werken GmbH & Co. KG

Ackerstr. 1,
47269 Duisburg, Germany
Tel +49 (203) 99 269-0 - Fax +49 (203) 29 92 83
www.hagerwerken.de