LARIDENT

MADE IN ITALY  
SINCE 1985

# CANNULA "ZERO"

CE

### A - SUP. BUCCALE MASCELLARE DX MAXILLARY RIGHT BUCCAL FACE

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## POSITIONING GUIDE

This guide is to assist you during ultrasonic scaling.

1. The "Cannula Zero" saliva ejector, which is called this because when used it eliminates over 95% of the generated aerosol, thanks to the flow rate of over 300 litres/minute, it is made up of a double mirror, double suction and selector.
2. The selector allows you to select the side from which you prefer to work (without subjecting your wrist and elbow to inadequate movements and positions) and position it correctly in the mouth so as to divert the suction flow from one of the two sides of the saliva ejector and to close the opposite one to avoid that, once positioned, it could create a suction effect with many inconveniences for the patient.
3. Turning the selector to "O" (Open) opens the suction side from which you choose to work. The suction side "C" (Closed) which comes into contact with the mucous membrane or tongue is automatically excluded.
4. It can be a useful support for those that use GBT therapy, scalign, dentists, preparations or surgery, etc.

The material used is biocompatible, autoclavable, hypoallergenic, antitoxic and, above all, certified.

As for the accessories (flexible tube and connectors) they can be useful to improve movements during the use of the saliva ejector itself, for this reason we have created light and space-saving connectors to help easy movement.

The tube is easy to get, there are different types, and just select a light one (such as those used for oxygen therapy).

Available in 2 colours, black or white and supplied in a specific package with 3 spare mirrors.

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WATCH  
DEMO HERE

### IMPORTANT

When changing the viewing side of the mirror it is necessary to select the suction side by acting on the selector. Then you have to turn the selector in the direction where you want to open the cannula suction.

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Available double sided mirror diameter 22 mm  
or 24 mm

**LARIDENT**

MADE IN ITALY  
SINCE 1985

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa) - 16030 Tribogna (Genova)

Tel. 0185 938068 - Fax. 0185 938595

www.larident.it

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